

Designed with care

As the leader in mask technology, ResMed has partnered with pediatric health experts to bring you Pixi[™] – a pediatric mask designed with children and their families in mind.

A mask designed specifically for children

Uniquely constructed using the anthropometric data of children aged two and older, the Pixi pediatric mask is not merely a scaled down adult mask.

Taking into account the bone structure, facial characteristics and skin sensitivity of children, the mask is designed to minimize pressure on soft faces. The Pixi pediatric mask is made with both the child and their caregiver in mind.

Every feature is carefully considered to address and **improve therapy comfort**, **acceptance and overall experience for everyone involved**.

Confident that their child is being treated effectively, caregivers can be free to focus on emotionally supporting their child on their journey.



ResMed.com

ResMed




Technology highlight

Headgear designed especially for children*

- Three points of adjustment—the mask can be adjusted at the top, side and bottom straps, providing a greater range of adjustment to cater to children of various ages.
- **Out of sight**—the headgear is designed to sit away from the child's eyes and ears to reduce irritation and obstruction.
- Clear line of sight—not only can the child see clearly while wearing the mask, but caregivers are also reassured and comforted by being able to see their child's eyes and more of their face.

*Designed using the specific head and facial anthropometric data of children.

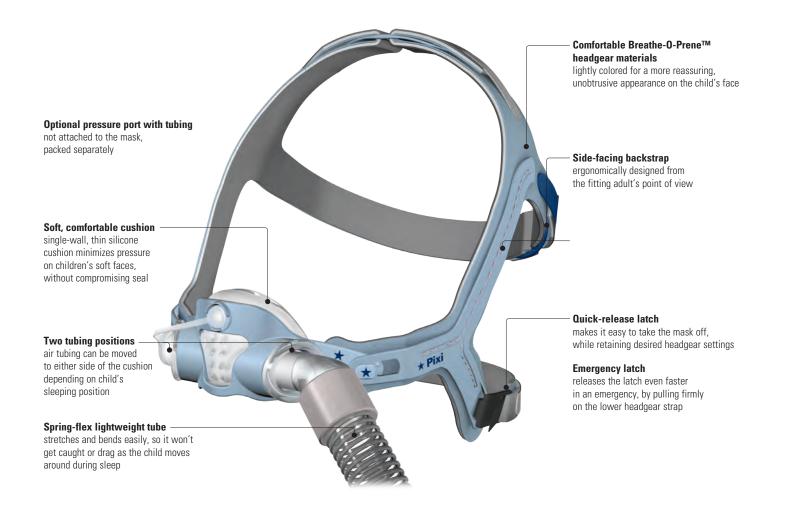



Fitting Guide



Pixi Hint: If fitting the mask for the first time, we recommend the child be sitting in an upright position to ensure correct fit.



Hold the mask gently against the • child's face, checking that the nose is fully enclosed by the cushion and the mouth is clear.



Attach the lower headgear strap • by holding the tab and hooking the guick-release clip onto the mask.

of the ears.



Pull the headgear over the head and move the middle strap down until it sits across the middle of the back of the head.



• Keeping your hand on the frame to maintain correct headgear position, bring the lower headgear strap behind the back of the neck.



- If necessary, adjust the straps by removing the Velcro® and gently tightening the headgear straps until comfortable. Do not overtighten.
- For optimal results, adjust the straps starting at the bottom strap and work upwards.

When fully fitted, ensure the top strap is not overtightened and is aligned with the top of the ears; the middle strap should sit across the back of the head aligned with the bottom



- Connect one end of the air tubing • to the swivel and the other end to the airflow device.
- Turn on the airflow device.




ResMed.com

ResMed Corp San Diego, CA, USA +1 858 836 5000 or 1 800 424 0737 (toll free). ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). See ResMed.com for other ResMed locations worldwide. Pixi and Stellar are trademarks of ResMed Ltd. Breathe-O-Prene is a trademark of AccuMED Technologies, Inc. ©2011 ResMed. Specifications may change without notice. 1014792/1 11 05

Tip



Components Guide

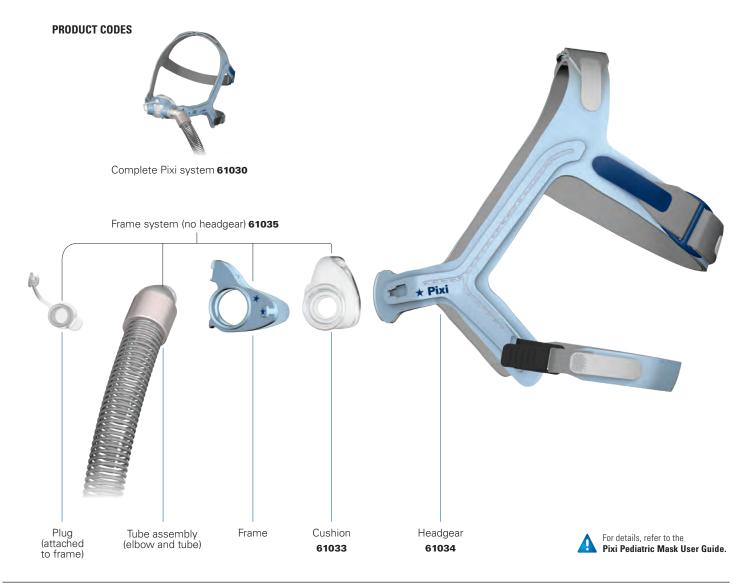

REIMBURSEMENT CODES

HCPCS	Code Descriptor
A7034	Nasal mask used with positive airway pressure device, 1 per 3 months
A7032	Replacement cushion for nasal mask, 2 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

Combine the Pixi mask

with the latest Stellar 100 or VPAP devices for a therapy system that delivers greater comfort, reliability and effective care.



Pixi Mask



ResMed.com