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Contents / Index / Índice

English	Introduction6
	Recommendations6
	Accessories Included7
	Spare parts description7
	Use instructions7
	Clean and maintenance7
	Technical specifications8
	Spare parts list9
	Trouble shooting guide9
	Symbols Key9
	Certifications10
Warranty11	

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Introduction

Congratulations for having purchased a Drive Piston Compressor Nebulizer.

Drive Medical warrants that this device has been produced using high quality materials and it complies with the safety rules in force both domestically and internationally.

Please read the instructions carefully before using the device and proceed accordingly.

“CAUTION: As per USA Federal regulations this product can only be sold under medical prescription.”

Recommendations

1- Read the manual carefully before using the device and keep it in case you need further information in the future.

2- This product was intended for aerosoltherapy only. Any other use will be considered wrong and consequently dangerous; Drive Medical will not be held responsible for damages caused by misuse or if the device has been used in electric fittings which do not comply with the safety rules in force.

3- Please take into account the following relevant points:

- The device is not protected from liquid penetration.
- The device should not be touched with wet or soaking hands.
- Keep the device in a safe place.
- Do not use this device in the presence of flammable substances.

4- Unplug the device before doing the cleaning or the maintenance operations. Unplug the unit when it is not being used.

5- Do not twist the jet nebulizer tubing during operation.

6- Do not leave solution in the jet nebulizer when it is not in use.

7- Do not pour more than 10 ml. of solution into the jet nebulizer.

8- Place the device on a flat surface.

9- Do not dip the device in water.

10- Use the unit intermittently. Maximum operating period: 30 minutes. Rest period: 30 minutes.

11- The accessories should not be shared by different patients to avoid infections.

12- Children may only use the device under adult supervision.

13- This device can only be repaired by an authorized Drive dealer.

NOTE: This device carries a thermal limiter which will shut off the unit to avoid overheating.

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Corriente alterna.....

Ver instrucciones.....

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1 Jet nebulizer, 1 Pediatric mask, 1 PVC tubing, 1 Adult mask, 1 Mouth piece, 3 Air Filters, 1 Instructions Manual and Warranty.

Spare parts description

See pictures on page 2.

Use Instructions

(see pictures on page 3)

- 1-** Take the unit out of the box.
- 2-** Press the red button and slide the cover backwards, then remove the accessories.
- 3-** Open the jet nebulizer (Pict. A) and pour the medication to nebulize into it, taking into account that the maximum capacity is 10 ml. (Pict. B). Twist both pieces and close it (Pict. C).
- 4-** Connect one end of the tubing to the jet nebulizer and the other end to the nebulizer outlet (Pict. D).
- 5-** Connect the mask (Pict. E) or the mouthpiece to be used to the jet nebulizer (Pict. F).
- 6-** Connect the unit to a suitable power receptacle.
- 7-** Turn the unit on pressing the ON/OFF switch to start the treatment.
- 8-** Turn the unit off and unplug it once the treatment has finished.
- 9-** Clean the jet nebulizer and the accessories as indicated in "Clean and Maintenance" section.
- 10-** Keep the jet nebulizer and the accessories into the corresponding compartment.

Clean and Maintenance

Important:

Clean the accessories before using the nebulizer for the first time or after each use following these instructions: wash all the pieces, except for the tubing, with warm water and detergent. Rinse with warm tap water for 30 seconds to remove the detergent. Dip the pieces into a clean bowl containing 3 parts of warm water and 1 part of vinegar during 30 minutes, disinfectant can also be used. Wash your hands and take out the pieces from the disinfectant solution, rinse them with warm tap water and leave them on an absorbent piece of paper. Keep the pieces in a plastic bag. Do not dry them with a towel. Clean the outer surface with a dry piece of cloth.

Warning: In order to avoid infections, it is highly recommended to clean the pieces after each use. Only nebulize solutions prescribed by a doctor should be used and follow his instructions carefully. Do not use balsam or aromatic substances.

- The jet nebulizer and the accessories must not be boiled. The jet nebulizer should be changed if it gets clogged.

Under no circumstances should the unit be opened. The nebulizer does not require either lubrication or maintenance.

Air Filter change

1- The air filter has to be replaced when dirty or damaged.

2- Remove the air filter by taking off plastic cap.

3- Turn plastic cap upside down to expose air filter.

4- With a tweezer, grab and remove filter.

5- Replace with a new filter

Caution: The nebulizer may get damaged if operated with a dirty or damaged filter. Do not replace with any material other than the correct filter.

Technical Specifications

Method of nebulization.....	Oilless piston compressor
Operating voltage.....	110V- 120V~ 60Hz
Class.....	<input type="checkbox"/>
Current.....	1.2A
Maximum jet nebulizer capacity.....	10 ml
Particle dimension (microns).....	smaller than 5 microns
Nebulized solution quantity.....	0.33 ml/min. or greater
Air flow.....	6-8 ltr/min.
Maximum pressure.....	32.85 psi
Weight.....	3.74 lbs / 1.7 kg
Device dimensions.....	10.4" x 4.9" x 3.7" 265 x 127 x 96 mm
Operating Temperature and Humidity	5° to 40° C, 30% to 95% RH 41°F to 104°F, 30% to 95% RH
Storage Temperature and Humidity.....	-10° to 40° C, 10% to 95% RH 14°F to 104°F, 10% to 95% RH
Atmospheric pressure range for transport and storage.....	500-1060 mb

*The specifications and the outer design are subject to change for improvement reasons without previous notice.

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See pictures on page 4

Trouble Shooting Guide

If a problem occurs during operation, please check and correct the following points:

Symptom	Possible cause	Suggested solutions
When switching it on, it does not work.	1- Not plugged in.	1- Check if the unit is properly connected.
The mist does not come out from the jet nebulizer, aerosol output reduced (low).	1- Not enough medication in the jet nebulizer. 2- Too much medication. 3- Tubing disconnected. 4- The jet nebulizer is defective. 5-The air filter is clogged.	1- Add medication, maximum capacity 10 ml. 2- Remove excess medication. 3- Connect the tubing. 4- Replace the jet nebulizer. 5- Replace the air filter.

Symbols Key

On.....		Class II.....	
Off.....	○	Type B.....	
AC.....	~	Read the instructions....	

Certifications

This product meets the highest international quality standards.

ISO 9001 Quality Systems (International)

ISO 13485 Medical Equipment (European Community)

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3 Year Limited Warranty

Drive Medical warrants to the original retail purchaser that this product is free from defects in material and workmanship for a period of 3 years from the date of purchase. This warranty does not cover failures resulting from misuse, abuse, alteration of the Drive Medical Nebulizer System, use of improper components or failure to comply with the operating instructions. This warranty does not include the Nebulizer System tubing or accessories.

THIS LIMITED WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, WHETHER ORAL, WRITTEN, EXPRESS OR IMPLIED. THERE ARE NO WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, EXCEPT FOR THOSE IMPLIED WARRANTIES ARISING BY OPERATION OF LAW WHICH ARE LIMITED IN DURATION TO THE 3 YEAR TERM OF THIS WARRANTY. DRIVE MEDICAL'S LIABILITY, WHETHER BASED ON CONTRACT, TORT, WARRANTY, STRICT LIABILITY, OR ANY OTHER THEORY, SHALL NOT EXCEED THE PRICE OF THE INDIVIDUAL DRIVE MEDICAL NEBULIZER SYSTEM WHICH IS THE BASIS OF THE CLAIM. IN NO EVENT SHALL DRIVE MEDICAL BE LIABLE FOR ANY LOSS OF PROFITS, FACILITIES, EQUIPMENT OR OTHER INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES.

Some states do not allow limitations on how long an implied warranty lasts, or allow the exclusion or limitation of incidental, consequential, or special damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights. You may also have other rights which vary from state to state.

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