EN DeVilbiss® IntelliPAP™ Positive Airway Pressure Device

DV54 AutoAdjust® CPAP Series

CAUTION—Federal (U.S.A.) law restricts this device to sale by, or on the order of a physician.

Assembled in USA of US & Imported Parts. Latex Free. Instruction Guide

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TABLE OF CONTENTS

| | |
|---|----|
| Symbol Definitions | 3 |
| Important Safeguards | 3 |
| Introduction..... | 5 |
| Intended Use/Indications for Use..... | 5 |
| Contraindications..... | 5 |
| What Is Obstructive Sleep Apnea (OSA)..... | 5 |
| How CPAP Therapy Works..... | 6 |
| Acclimating To Your Therapy..... | 6 |
| Benefits Of Your Therapy..... | 7 |
| Helpful Resources | 7 |
| Key Features..... | 8 |
| Unpacking The Contents | 8 |
| IntelliPAP Device..... | 8 |
| Keypad..... | 8 |
| System Assembly without Humidification..... | 9 |
| Keypad and Display | 9 |
| Operation | 10 |
| Start Up..... | 10 |
| Using The Comfort Delay Feature | 10 |
| Shut Down | 11 |
| Advanced Menu Controls | 11 |
| Advanced Menu List..... | 12 |
| IntelliPAP Enable Menu List..... | 13 |
| AutoAdjust Patient Menu | 14 |
| CPAP Patient Menu..... | 14 |
| Patient Messages | 15 |
| Reminders | 15 |
| Notifications | 15 |
| IntelliPAP Travel Information..... | 16 |
| IntelliPAP DC Operation..... | 16 |
| Battery Power | 16 |
| Supplemental Oxygen | 16 |
| Accessories/Replacement Items..... | 18 |
| Maintenance | 19 |
| Cleaning..... | 20 |
| Troubleshooting..... | 21 |
| Specifications | 23 |
| Warranty..... | 24 |

SYMBOL DEFINITIONS

- | | | | |
|--|--|--|--------------------------------------|
| | Class II electrical protection—double insulated | | Type BF equipment— applied part |
| | Data Port Input/Output | | Standby – turns the blower on or off |
| | Next Item on LCD display | | Increase value shown on LCD display |
| | Previous Item on LCD display | | Decrease value shown on LCD display |
| | Attention - Consult Accompanying Documents | | Heat |
| | DC Input: 12 VDC, center pin is positive | | 100 – 240 Volts AC input 50/60 Hz |
| | Locked | | Unlocked |
| | IPX1 IPX1 Drip-Proof, Vertical | | |
| | Delay – starts the delay cycle in CPAP mode, if prescribed – reduces pressure to lower pressure limit in AutoAdjust mode | | |
| | This device contains electrical and/or electronic equipment that must be recycled per EC Directive 2002/96/EC - Waste Electrical and Electronic Equipment (WEEE) | | |

IMPORTANT SAFEGUARDS

When using electrical products, basic safety precautions should always be followed. Read all instructions before using this device. Important information is highlighted by the following terms.

- DANGER** Urgent safety information for hazards that will cause serious injury or death.
- WARNING** Important safety information for hazards that might cause serious injury.
- CAUTION** Information for preventing damage to the product.
- NOTE** Information to which you should pay special attention.

PLEASE READ ALL INSTRUCTIONS BEFORE USING THIS DEVICE.

SAVE THESE INSTRUCTIONS

DANGER

Electric Shock Hazard – Do not use while bathing.

Electric Shock Hazard – Do not immerse this device into water or any other liquid.

Electric Shock Hazard – Do not attempt to open or remove the enclosure. There are no user-serviceable internal components. If service is required, return the product to your home care provider. Opening or tampering with the product will void the warranty.

WARNING

The DeVilbiss IntelliPAP™ should be used only with masks recommended by DeVilbiss, your physician or respiratory therapist.

To avoid rebreathing of exhaled air, do not use a CPAP mask unless the device is turned on and providing a supply of air. Venting in the mask should never be blocked. When the device is turned on and providing a fresh supply of air, exhaled air is flushed out of the mask vent. However, when the device is not operating, exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most CPAP devices.

The DeVilbiss IntelliPAP is not a life support device and may stop operating with certain device faults or with a power failure. It is intended to be used on spontaneously breathing individuals weighing 66 lbs/30 Kg or greater.

To avoid electric shock, always unplug power cord from wall outlet power source when performing cleaning.

Use only accessories recommended by DeVilbiss.

CAUTION—*The circular data port connector located on the back of the IntelliPAP is used to attach accessories to the device. The connector must only be used with accessories approved for use by DeVilbiss. Do not attempt to attach any other device to this connector as it may damage the CPAP or the accessory device.*

CAUTION—*Never rinse or place the device in water. Never allow liquids to get into or around any of the ports, switches or air filter; doing so will result in device damage. If this occurs, discontinue use and remove the power cord from the power source. Allow the device to completely dry before use.*

CAUTION—*Do not place the IntelliPAP where it can be bumped onto the floor or where the power cord may create a trip hazard.*

CAUTION—*Only the DeVilbiss DV5 series Heated Humidifier system is recommended for use with the IntelliPAP. Other humidifier systems may prevent the device from detecting snoring and may cause inappropriate pressure levels in the mask.*

CAUTION—*Oxygen is a prescription gas and should only be administered under the supervision of a physician.*

INTRODUCTION

Intended Use/Indications for Use

The DeVilbiss® IntelliPAP™ Model DV5 Series is intended for use in treating obstructive sleep apnea in patients 66 lbs/30 kg and above.

Contraindication

Do not use IntelliPAP if your upper airway has been bypassed.



NOTE—Symptoms of dryness of the throat, nasal passages, and the mouth are common with positive airway pressure therapy. This is especially true in dry climates and during the cold season when humidity in the air is typically lower than at other times. The IntelliPAP features an optional humidifier system to help minimize these effects.

What Is Obstructive Sleep Apnea (OSA)?

Obstructive Sleep Apnea (OSA) is a breathing disorder that affects more than 22 million Americans – approximately 4% of middle-aged men and 2% of middle-aged women.

Physiological factors such as size of tonsils, shape of the nose, narrowing of the upper airway, or any combination of these can contribute to the potential for having OSA. During sleep, your airway can relax so much that it begins to collapse and becomes obstructed. Relaxation allows the tissue at the back of your throat to vibrate as you breathe causing snoring. But as your airway collapses more fully, it blocks your airflow and stops your breathing altogether. (In fact, the word “apnea” is derived from a Greek term meaning “without breath.”)

This stoppage occurs repeatedly throughout the night, each episode lasting from just a few seconds to more than a minute. As your blood oxygen level drops to dangerous levels, your central nervous system triggers a sudden gasp for breath, partially awakening you and preventing you from experiencing the sustained, deep sleep your body needs. As a result, it is common to feel fatigued, stressed and irritable the following day – a less-than-optimal and potentially dangerous condition in which to work, drive, and make decisions.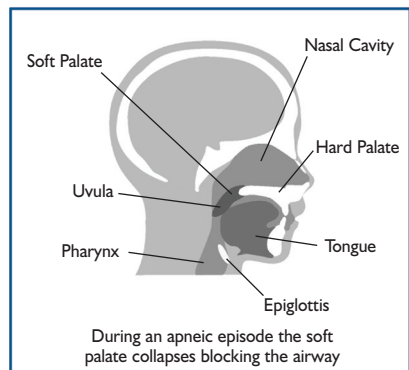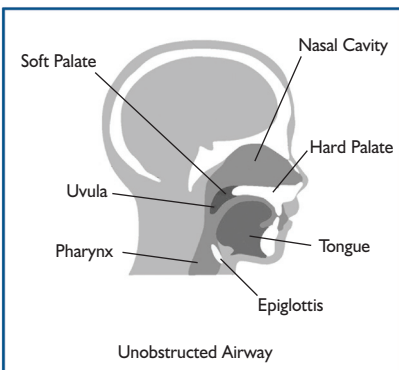

There are important medical concerns as well. OSA starves your heart, brain, and organs of life-sustaining oxygen. Over time, this deprivation can seriously impact your health

potentially resulting in memory loss, impotence, hypertension, coronary disease, strokes, and heart attacks. Recent studies indicate that OSA has the same risk factor for heart disease as smoking, high cholesterol and alcohol, and estimate that as many as 38,000 people a year die from the effects of OSA.

Clearly, OSA is a serious medical condition that requires attention and treatment.

How CPAP Therapy Works

There are several treatments for OSA including surgery and CPAP therapy. However, CPAP therapy has proven very effective for most patients. CPAP stands for “continuous positive airway pressure” which is exactly what the DeVilbiss IntelliPAP device delivers to your upper airway, acting as an “air splint” to keep the passage open while you sleep.

This reduces or eliminates the obstruction, allowing you to enjoy the deep, refreshing, uninterrupted sleep you need. More importantly, your body will get the rest and oxygen it needs with more continuous, uninterrupted sleep – helping you avoid some of the serious health risks associated with OSA. And the IntelliPAP is designed to provide this therapy to you quietly and comfortably so that you can sleep through the night without distraction.

The DeVilbiss IntelliPAP AutoAdjust® system automatically adjusts air pressure by measuring your snoring and breathing patterns on a breath-by-breath basis, and compensating to meet your pressure needs. Your breathing patterns are continuously analyzed, stored and used to make any needed pressure adjustment.

The rate of pressure adjustment can be as high as 10 cmH₂O in a 10 minute period with changes made gradually enough to prevent disturbing your sleep yet quickly enough to respond to your pressure needs. In the absence of respiratory events, the pressure is reduced to maximize your comfort. The IntelliPAP AutoAdjust is also capable of operating in CPAP mode – without the automatic functionality described above – to give your physician maximum versatility in prescribing treatment for your sleep condition.

Acclimating To Your Therapy

Naturally anything new – including CPAP therapy – takes some getting used to. Having your mouth or nose (or both) covered by a mask may make you feel anxious or claustrophobic. Feeling the rush of air from your IntelliPAP may cause discomfort or concern. But don't worry – every successful, long-term CPAP user began where you are today with the same reactions and concerns, and overcame them to enjoy the full benefits of their therapy.

Here are some steps to help you gradually ease into your therapy and overcome your anxieties:

1. Try attaching the mask to the CPAP unit, and switching the unit “On.” Practice breathing through the mask for one hour while watching television, reading, or performing some other sitting activity.
2. If you nap, use the CPAP during this time.
3. Use CPAP at least during the first initial 3-4 hours of nighttime sleep.
4. Use CPAP through an entire night of sleep.

Advance through the above steps one step at a time and progress to the next step as soon as you can carry out the current step without anxiety.

Benefits Of Your Therapy

As a DeVilbiss IntelliPAP user, you are on your way to living a happier, healthier life – a life free from the sleep disruptions and health risks associated with OSA. After you become acclimated to CPAP therapy, you may be surprised at how much better you feel, both physically and emotionally. CPAP therapy may give you the opportunity to finally experience the deep, restful sleep you've been missing out on. Many patients find they have more energy, better judgment, and improved concentration during the day. With effective therapy you'll be less stressed, less irritable and more like your old self again, with a healthy, positive outlook on life. In fact, you may well wish you'd gotten your IntelliPAP sooner!

Helpful Resources

As you begin your IntelliPAP therapy, remember: you're not in this alone! You're part of a large and growing network of CPAP users, and many resources are available to answer your questions, address your concerns, and help make your therapy as effective and effortless as possible.

American Academy of Sleep Medicine

One Westbrook Corporate Ctr, Ste 920
Westchester IL 60154
Phone: (708) 492-0390
Fax: (708) 492-0943
www.aasmnet.org

American College of Chest Physicians

3300 Dundee Rd
Northbrook, IL 60062-2348
Phone: (847) 498-1400 or (800) 343-2227
Fax: (847) 498-5460
www.chestnet.org

The Sleep Apnoea Trust

12a Bakers Piece,
Kingston Blount
Oxon.
OX39 4SW
Tel: +44 845 60 60 685
www.sleepapnoea-trust.org

National Heart, Lung & Blood Institute Information Center

P.O. Box 30105
Bethesda, MD 20824-0105
Phone: (301) 592-8573
Fax: (240) 629-3246
www.nhlbi.nih.gov

American Academy of Neurology

1080 Montreal Avenue
St. Paul, Minnesota 55116
Phone: (651) 695-2717 or (800) 879-1960
Fax: (651) 695-2791
www.aan.com

American Sleep Apnea Association

1424 K Street NW, Suite 302
Washington, DC 20005
Phone: (202) 293-3650
Fax: (202) 293-3656
www.sleepapnea.org

National Center on Sleep Disorder Research National Heart Lung & Blood Institute (NIH)

6701 Rockledge Drive
Bethesda, MD 20892
Phone: (301) 435-0199
Fax: (301) 480-3451
www.nhlbi.nih.gov/sleep

National Sleep Foundation

1522 K Street, NW, Suite 500
Washington, DC 20005
Phone: (202) 347-3471
Fax: (202) 347-3472
www.sleepfoundation.org

KEY FEATURES

Unpacking The Contents

The following items are included in your DeVilbiss IntelliPAP system:



- | | |
|----------------------|----------------------|
| 1. IntelliPAP device | 5. Air supply tubing |
| 2. Spare filter pack | 6. User manual |
| 3. AC line cord | 7. DVD |
| 4. Carrying case | |

If any of the above items are missing, please contact your home care provider.

NOTE—If you received a humidifier with your IntelliPAP system, please refer to the humidifier’s instruction guide for additional information.


DeVilbiss IntelliPAP Device (Figures A and B)

- Keypad/LCD Display (see Figure C)
- Air Supply Port on Back
- Air Supply Port on Bottom (for optional humidifier)
- Air Supply Port Plug
- Heater Power Connector (for optional humidifier)
- AC Power Connector
- DC Power Connector
- Data Port
- Power Cord (appropriate to your wall outlet) (not shown)
- Air Inlet Filter Opening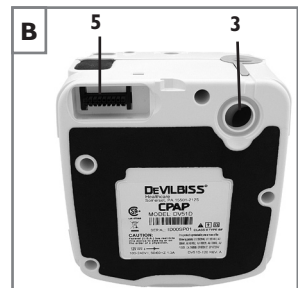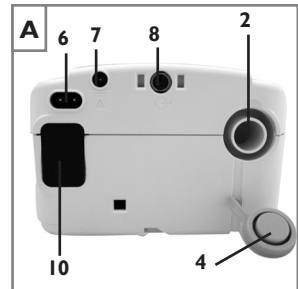

Keypad (Figure C)

- On/Off
- Previous Item
- Next Item
- Delay
- Decrease Value
- Increase Value
- Heater Power LED (for optional humidifier)

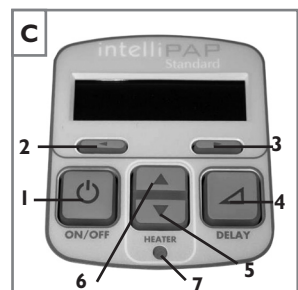Ask your home care provider for information about other DeVilbiss masks, equipment, and accessories.

SYSTEM ASSEMBLY WITHOUT HUMIDIFICATION

NOTE—If your system includes a humidifier, please refer to the instructions provided with the humidifier for assembly and operation.

1. Locate the air supply port and heater accessory connector on the bottom of the IntelliPAP device. Ensure that each opening has the appropriate soft cover securely attached.
2. Place the IntelliPAP on a stable surface such as a nightstand or table. If you wish, you may also place the device on the floor at the bedside, being careful to place the unit where it won't be kicked or stepped on. Ensure that the air inlet in the back of the device is not blocked by anything such as curtains or bedding.


NOTE—Never place the IntelliPAP system on a soft surface such as a bed or couch during operation.

CAUTION—The height of the IntelliPAP system must always be lower than the interface mask when using a humidifier to prevent water from getting into the CPAP mask.

3. Insert the power cord into the back of the device. Make sure the power cord is fully inserted into the connector. Plug the other end of the power cord into the AC wall outlet.
4. Attach the air supply tubing to the air supply interface in the back of the device.
5. Prepare the mask (supplied by your home care provider) by referring to the mask's instruction guide.
6. Connect the air supply tubing to the mask.

NOTE—The air supply tubing supplied with the device is specially designed to prevent kinking and allow you to receive smooth airflow. Use only the 22 mm diameter, smooth-bore, kink-resistant tubing supplied with this device or an air supply tubing recommended by your home care provider.

Keypad And Display

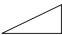
| Basic DeVilbiss IntelliPAP Controls | | |
|-------------------------------------|---|--|
| Name | Symbol | Function |
| ON/OFF |  | The ON/OFF button starts and stops the flow of air from the device. |
| HEATER |  | If your IntelliPAP has a heater/ humidifier attached, these buttons adjust the heater setting. Increase the setting for more humidity and decrease the setting for less humidity. |
| DELAY |  | The Delay button activates the comfort delay feature and only functions if this feature has been enabled. To understand how to set the parameters of this feature, please refer to the Using the Comfort Delay Feature found on page 10. |

NOTE—The additional control buttons that you see on the IntelliPAP control panel provide advanced controls for specific settings. Please refer to the 'Advanced Menu Controls' section on page 11 for more details.

OPERATION

Startup


To begin operation, verify that your IntelliPAP is connected to a 'live' power source. When plugged in, the LCD display should appear as noted.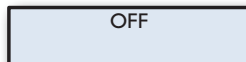


Put on your CPAP mask and press the ON/OFF button on the keypad to turn on airflow OR put on your CPAP mask and take a few breaths to turn on airflow automatically. The pressure in your mask will rise to the set pressure in about 10 seconds.

NOTE—If the device does not automatically start after taking 2-3 breaths, this feature may have been disabled. If Auto ON/OFF is disabled, you must manually press the ON/OFF button to start and stop the device.

The keypad display shows the actual mask pressure while the device is in use. Because this is the actual pressure, the device pressure reading may vary slightly as you breathe. Your display should appear as noted.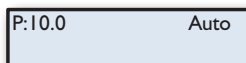

If your DeVilbiss IntelliPAP system includes a humidifier, its heater controls and status are displayed as noted in the image.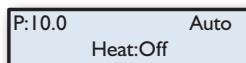


NOTE—If the display on your device is not similar to those listed above, refer to the chapter section titled "Patient Messages" on page 15.

Using The Comfort Delay Feature

Your IntelliPAP has a comfort delay feature to help make it easier for you to fall asleep. The airflow will start at a low pressure and then gradually ramp-up to the prescribed setting over the last few minutes of the specified delay time.

NOTE—When the delay comfort feature is activated, a message will flash on the display periodically, indicating the amount of time remaining in the delay.

You can adjust the amount of time spent in comfort delay by using the advanced device controls. The Delay Time can be adjusted from 0 to 45 minutes in 5-minute increments. Refer to page 12 for details.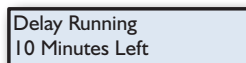

NOTE—If you are unable to change the delay time, your provider has locked this setting. Refer to the keypad menu guide on page 12 for details.

Comfort delay is available in both IntelliPAP Standard and IntelliPAP AutoAdjust models, and in all modes of operation.

Comfort Delay Feature In Standard CPAP Mode

When you use the comfort delay feature while your device is operating in CPAP mode, the airflow you receive at the beginning of your therapy is temporarily lowered to a pressure selected by your physician when the device was setup.

Press the delay button to activate the comfort delay feature. The airflow pressure will drop to the prescribed comfort delay pressure.

In the final 10 minutes of the delay period, the pressure will gently rise to the pressure prescribed for your therapy. You may repeat the comfort delay by pressing the delay button again if you have not fallen asleep during the first use.

Comfort Delay Feature in AutoAdjust Mode

When you use the comfort delay feature while your IntelliPAP is operating in AutoAdjust mode, the airflow you receive at the beginning of your therapy is the lowest pressure prescribed by your physician when the device was setup.

You do not need to activate the comfort delay; it will turn on automatically. The airflow pressure will rise only to the low pressure setting and remain there for the duration of the delay period. The AutoAdjust will not adjust the pressure setting during the comfort delay period, allowing you to fall asleep without responding to early respiratory events.

Shut Down

Press the ON/OFF button to turn the device off. You may also simply remove your mask. After a few seconds the device will display the message “Mask Off. Please check mask fit.” If you do not put the mask on again the device will automatically turn off in about 20 seconds. Pressing any button will remove the message from the display.

NOTE—If the DeVilbiss IntelliPAP does not turn off automatically, the feature may have been disabled. In this case you must manually turn the device off.

ADVANCED MENU CONTROLS

With the device on, press the “Next” or “Previous” buttons on the keypad to scroll through the items on the display. Not all of the features in the table below are available in all models; some are optional and some depend on what your physician and/or home care provider has selected. The table shows the items you may see, and describes the feature associated with each item. Also, some features are only available in AutoAdjust Mode or with the DeVilbiss SmartLink™ Therapy Management Module – these are noted in the ‘Mode’ column below. If you do not see some of these options, your model or configuration does not support them.

| Name | Symbol | Function |
|----------------|--------|---|
| PREVIOUS ITEM | | Pressing this button displays the previous menu option. |
| NEXT ITEM | | Pressing this button displays the next menu option. |
| SETTING CHANGE | | These buttons allow you to adjust the setting that is shown on the display. |

Please note that the values displayed are examples only.

Advanced Menu List

| Display Example | Mode | Description |
|-----------------|-----------------|---|
| | All | This is the default screen when the IntelliPAP is turned on. The actual mask pressure and heater setting are shown. The displayed pressure may vary slightly while breathing. If the heater system is not connected, the heat setting will be blank. |
| | AutoAdjust Mode | This is a prescription setting which limits the maximum pressure at which the IntelliPAP will operate if in AutoAdjust mode. Your home care provider programs this setting for you during device setup. |
| | AutoAdjust Mode | This is a prescription setting which sets the minimum pressure at which the IntelliPAP will operate in AutoAdjust mode. Your home care provider programs this setting for you during device setup. |
| | CPAP Mode | This is a prescription setting which controls the operating pressure. |
| | All | Selectable delay time setting. Your provider has set the delay time. If the unlocked symbol is shown, you are able to adjust this setting. If the locked symbol is shown, this setting can only be adjusted by your provider. |
| | CPAP Mode | This is a prescription setting which controls the operating pressure during the delay time. |
| | All | The standard tubing length supplied with the device is 6 feet (1.8 meters). However, you can adjust your IntelliPAP to accommodate an air supply tubing length of 10 feet (3.0 meters). Replacement tubing can be obtained through your home care provider (part numbers listed at the end of this manual). |
| | All | Pressing the down key enters the Enable menu. Refer to next section for Enable Menu details. |
| | All | This setting changes the contrast for the LCD display. This only needs to be adjusted if you find it difficult to see the display from your viewing angle. |
| | All | The total number of hours (in 1/10 hour increments) that the IntelliPAP has been providing therapy. |
| | All | The total number of hours (in 1/10 hour increments) that the IntelliPAP has been in operation. |

| Display Example | Mode | Description |
|--------------------------------------|------|---|
| Firmware Version V0.01 01/06/2006 | All | Shows version of installed firmware. |
| Serial Number HD123456 | All | Shows the serial number of the device. |
| SmartCode FF28D9 | All | SmartCode™ is used for therapy management reporting; therapy information is encoded in a series of letters and numbers, and includes a special character for error checking to ensure accuracy. |

Intellipap Enable Menu List

The Enable menu is available in all modes and is a sub-menu that allows you to control various functions of your IntelliPAP. It is accessed by pressing the down button while in this Advanced Menu.


| Display Example | Description |
|---------------------------|---|
| Auto-ON Enabled | This option controls the Auto-ON feature, which automatically starts the flow of air after breathing once or twice into the mask. |
| Auto-OFF Enabled | This option controls the Auto-OFF feature, which automatically stops the flow of air after taking off the mask. The device may require up to 30 seconds to turn OFF depending on the kind of mask used. |
| Mask Fit Check Enabled | This option controls the mask fit check feature. The air flow to your mask is constantly measured and if the amount of airflow exceeds 95 liters/minute for more than 10% of the time used, a notification will be created. This notification will be displayed the next time you use the IntelliPAP. |
| Exit Menu Press ▲ | Pressing the up key exits the Enable menu. |

DV54 AutoAdjust Patient Menu Layout

The following chart shows by Mode each Display you will scroll through using the Next or Previous buttons on the keypad. Pressing the Down key at the Enable Menu Display will allow you to enter this submenu - then use Next or Previous buttons to scroll through it; press the Up button to return to main menu. Once the final Display is reached, pressing the Next button will return you to the first Display.

MENU - AutoAdjust Mode

MENU - CPAP Mode

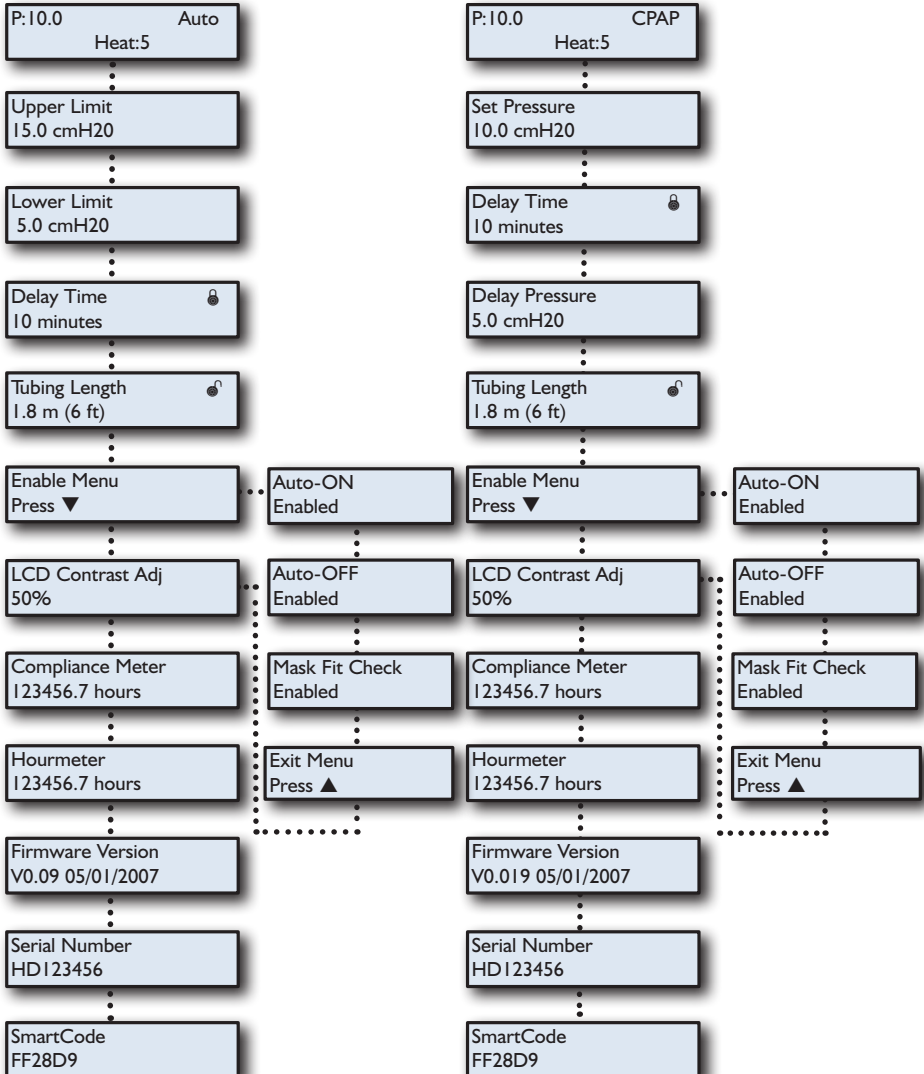PATIENT MESSAGES

Your IntelliPAP will alert you of issues that may require some action on your part in order to make your therapy more effective. These messages are displayed when the device is turned on, either by pressing the ON/OFF button or by breathing into the mask. You can press any key or button to clear the message from the display, but the IntelliPAP will operate normally even if the message is not cleared.

Two kinds of messages are displayed: Reminders and Notifications.

Reminders

Reminders alert you when parts of your system need to be replaced. Some components of your system wear over time and, if not replaced, may compromise your therapy. If you see a message on the display that is not listed below, you may have the optional DeVilbiss SmartLink™ Module attached to your device, which provides additional messages. Refer to the documentation that was provided with the SmartLink Module.

Clean Filter – The filter should be checked every 10 days for signs of dirt or wear so that it can be cleaned as needed. This message helps to remind you to check it regularly.

Notifications

Notifications identify conditions in your IntelliPAP that may require action by you or your home care provider in order to maintain a high level of therapy. If you see a message on the display that is not listed below, you may have the optional DeVilbiss SmartLink Module attached to your device, which provides additional messages. Refer to the documentation that was provided with the SmartLink Module.

NOTE—The values displayed are examples only.

Delay Running xx Minutes Left – While a comfort delay is active, this message is flashed on the display every 5 seconds to let you know how much time is left in the delay.

Mask Leak – This message means the IntelliPAP has detected a large amount of airflow during therapy for at least 10% of the time during your previous use session. This notification is displayed when the device is turned on. If this message is displayed, put on the mask and adjust the headgear to ensure the mask is properly fitted to your face. Follow the mask manufacturer's directions for adjusting the mask and headgear for proper fitting. Available on AutoAdjust models.

Mask Off – This message appears whenever a high airflow condition occurs. High airflow is normally due to a poor mask fit or a removed mask. Check for leak around the mask seal and make adjustments as necessary according to the mask manufacturer's instructions. This message will be displayed until the high airflow problem is corrected. If the high airflow condition persists for about 20 seconds, the device will automatically turn off (if AutoOFF is enabled).

Device Fault – If a device fault message is displayed, refer to Troubleshooting for instructions.

INTELLIPAP TRAVEL INFORMATION

International Power Changes

Your DeVilbiss IntelliPAP is equipped with a universal power supply. This means that it is automatically capable of accepting line voltages of 100 to 240V~ 50/60Hz. You do not need to make any adjustments to the device if you are traveling to another country where the line voltage is different than your domestic line voltage. Simply contact your home care provider for the correct style power cord for the region in which you will be traveling (refer to Accessories/Replacement Items for the correct part number).

Altitude

Your IntelliPAP automatically compensates for changes in altitude between sea level and 9000 ft (2750 m). You do not need to make adjustments to the device for changes in altitude.

INTELLIPAP DC OPERATION

Battery Power

Your IntelliPAP can be operated from a 12V DC power source in a motor home, boat or other recreational vehicle. The power source can be used with a 12V DC accessory cable or a 12 V DC stand-alone battery. Both parts are available from your home care provider (refer to Accessories/Replacement Items for the correct part numbers). Both accessory cables are required for stand-alone battery operation. The stand-alone battery connection uses clamps that attach to the terminal connections of the battery.

NOTE—The heater for the IntelliPAP's humidification system will not operate directly from a 12V DC power source. If you need humidification, passover humidification is still available. Alternately, instead of connecting 12V DC power directly to the IntelliPAP you can pass the DC power through an inverter that will deliver AC power to the system. This will allow the heater to function. The power rating of the inverter must be at least 200 watts @ 110V~ / 400 watts @ 220V~.

NOTE—The IntelliPAP will automatically switch power sources between AC and 12V DC if both sources are available. If your unit is connected to both AC and 12V DC power sources, it will operate on AC power as long as AC power is present, and automatically switch to 12V DC power if AC power is lost. When AC power is restored, it will return to the AC source. You do not need to reset or adjust any settings when the power source switches, but remember that your heater will not operate on 12V DC.

SUPPLEMENTAL OXYGEN

Supplemental oxygen is sometimes prescribed in addition to CPAP for treatment of Obstructive Sleep Apnea. If your physician prescribed oxygen for your sleep therapy, you can add the oxygen in one of two ways:

1. Obtain the optional oxygen adapter (part #7353D-601) from your home care provider and connect it to the outlet port on the IntelliPAP. Connect one end of the air supply

tubing to the open end of the adapter and the other end of the air supply tubing to the mask.

2. If your mask is equipped with an oxygen port, your physician may request that you apply oxygen directly to your mask.

WARNING-OXYGEN USE

Oxygen supports combustion. To avoid possible physical injury, do not smoke while using this device with supplemental oxygen. Do not use this device near hot objects, volatile substances or sources of open flames.

Always turn on the IntelliPAP before turning on the oxygen source. Turn the oxygen source off before turning off the IntelliPAP.

This device is not suitable for use in the presence of a flammable anesthetic mixture with air or oxygen, or nitrous oxide.

At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration will vary depending on the pressure setting, patient breathing pattern, mask selection, and leak rate. This warning applies to most types of CPAP devices.

Never allow the oxygen source to run continuously while connected to the IntelliPAP if the device is not in use. If the IntelliPAP is not being used, turn off the oxygen flow.

CAUTION—*Oxygen is a prescription gas and should only be administered under the supervision of a physician.*

CAUTION—*The flow setting of the oxygen source must be specified by a physician.*

ACCESSORIES/REPLACEMENT ITEMS

Replacement Items

| | |
|--|-----------|
| Air-inlet filter (4/pk) | DV51D-602 |
| Optional fine particle filter (4/pk) | DV51D-603 |
| Air supply tubing (6') | 7351D-616 |
| Air supply plug | DV51D-604 |
| Heater connection cover | DV51D-605 |
| Oxygen adapter..... | 7353D-601 |
| Carrying case | DV51D-610 |

Power Items

| | |
|-----------------------------------|-----------|
| AC power cords | |
| USA | DV51D-606 |
| Europe, except U.K. | DV51D-607 |
| U.K. | DV51D-608 |
| Australia | DV51D-609 |
| DC power cord | DV51D-619 |
| DC battery clamp-on adapter | DV51D-696 |

Masks Recommended For Use With The DeVilbiss IntelliPAP:

| | |
|--|--------|
| DeVilbiss FlexSet® Mask | 9354D |
| DeVilbiss FlexSet® Shallow Mask..... | 9354S |
| DeVilbiss FlexSet® Gel Mask..... | 9354G |
| DeVilbiss FlexSet® Gel Shallow..... | 9354GS |
| DeVilbiss FlexAire® Mask..... | 9353D |
| DeVilbiss FlexAire® Shallow Mask..... | 9353S |
| DeVilbiss Serenity® Mask..... | 9352D |
| DeVilbiss Serenity® Shallow Mask | 9352S |
| DeVilbiss Serenity® Gel Mask | 9352G |
| DeVilbiss Serenity® Gel Shallow | 9352GS |

Optional Heated Humidifier

| | |
|--|-------|
| DeVilbiss Heated Humidifier (cradle and chamber) | DV5HH |
|--|-------|

INTELLIPAP MAINTENANCE

DANGER

Electric shock hazard – Do not attempt to open or remove the enclosure; there are no user-serviceable components inside. If service is required, contact your home care provider for instructions on obtaining service of your device. Opening or attempting to service your device will void the warranty.

WARNING

If the operation of the device appears to deviate from normal operation or begins to produce unusual sounds, discontinue use and contact your home care provider for assistance.

If the device has been dropped, verify functionality of the device. If the device does not function or the enclosure is damaged, discontinue use and contact your provider.

If the device does not appear to be functioning properly refer to the Troubleshooting section for instructions.

Pressure Accuracy—The IntelliPAP is designed to provide accurate pressure performance from the factory without additional calibration. No routine service is required for a period of 2 years of use, provided the device is used in accordance with the manufacturer's directions. After 2 years the device should be inspected.

NOTE—*Some countries require periodic servicing and calibration of this kind of medical device. Contact your home care provider for further information.*

Standard Air-Inlet Filter—Check the filter every 10 days and clean it if necessary. Replace the filter every 6 months or sooner if it is damaged. The standard filter is designed to filter particles larger than 3.0 microns in size. The IntelliPAP **MUST** have the standard filter in place during operation.

Optional Fine Particle Filter—Check the filter every 10 days, and replace it if it is dirty or damaged; otherwise replace the filter every 30 days. The optional fine particle filter is designed to filter particles as small as 0.3 microns in size and larger.

Filter Maintenance—Refer to Cleaning for instructions.

INTELLIPAP CLEANING

WARNING

To avoid electric shock, always unplug the power cord from the wall outlet power source before cleaning.

CAUTION—Never rinse or place the device in water. Never allow liquids to get into or around any of the ports, switches or air filter; doing so will result in product damage. If this occurs, do not use the device. Contact your home care provider for service.

Device Cabinet

1. Unplug the IntelliPAP from the power source. Wipe the enclosure with a clean, damp cloth every few days to keep the enclosure dust free.
2. Allow the device to dry completely before plugging it into a power source.

Filters

Air-Inlet Filter (Standard)

1. The air-inlet filter should be checked every 10 days and cleaned as necessary. The filter is located in the rear of the device.

CAUTION—Proper filter function is important for the operation of the device and to protect the device from damage.

2. To clean, remove the dark outer foam filter from the frame.
3. Wash the filter in a solution of warm water and dishwashing detergent, and rinse with water. Allow the filter to dry completely. If the filter (part #DV51D-602) is damaged, contact your home care provider for a replacement.
4. Ensure the filter is completely dry before installation and use.

Fine Particle Filter (Optional)

1. If you have an optional fine particle filter (part #DV51D-603), inspect this filter and replace it if it shows signs of dirt or damage. Do not wash.
2. Install the fine particle filter behind the standard filter.

NOTE—If the fine particle filter is not installed first according to the directions, the life of the filter will be reduced requiring more frequent replacement.

Tubing

The air supply tubing should be cleaned daily. Remove the tubing from the device and mask. Use a mild detergent and water to clean the inside of the tubing. Rinse the tubing and allow to air dry.

Mask and Headgear

Clean the mask and headgear according to the manufacturer's instructions.

TROUBLESHOOTING

DANGER

Electric Shock Hazard - Do not remove the device cabinet; there are no user-serviceable internal components. The cabinet should only be removed by a qualified DeVilbiss provider.

If you are having trouble with the operation of your DeVilbiss IntelliPAP, please refer to the table below for a list of problems with possible causes and remedies. If you are unable to resolve the problem by following the instructions, contact your home care provider for assistance.

| Problem | Probable Cause | Remedy |
|---|---|--|
| Nothing shows on the display. | <ol style="list-style-type: none"> 1. Device is not plugged in or the power cord is not fully inserted. 2. There is no outlet power. | <ol style="list-style-type: none"> 1a. Verify that power cord is firmly connected to the IntelliPAP and the power outlet. 1b. If you are using a DC power source make sure the cable connections are secure. Verify that your battery is charged. 2. Locate a suitable functioning power source. |
| The IntelliPAP does not start when breathing into the mask. | <ol style="list-style-type: none"> 1. The Auto-ON feature has been disabled. 2. There is no power to the device. 3. Breath is not deep enough for the Auto-ON feature to detect. 4. You are using a full face mask that has an anti-asphyxia valve. 5. The air supply port plug is missing or not fully inserted. 6. The humidifier chamber is not fully engaged into the cradle or is missing. | <ol style="list-style-type: none"> 1. Use the ON/OFF button to start and stop the device. 2. Verify that the power cord is firmly connected to the device and the power outlet. 3. Breathe deeply in and out to start the IntelliPAP 4. Auto-ON may not work because your exhaled breath escapes through the open valve. Use the ON/OFF button to start and stop the device. 5. Make sure the air supply port plug is fully inserted into the unit. 6. Slide the humidifier chamber fully into the slot. Verify the latching lever snaps into place. |
| The airflow has stopped unexpectedly during use. | <ol style="list-style-type: none"> 1. The Auto-OFF feature has detected large airflow because of loosely fitting mask. 2. During use your mouth opens and you begin to mouth breathe. | <ol style="list-style-type: none"> 1. Ensure you have a good mask fit; adjust the mask and headgear as needed. 2. Contact your home care provider for a chin strap or different mask which will help prevent mouth breathing. <p>If problem is not resolved with above remedies, disable Auto-OFF feature in Advanced Menu options.</p> |

| Problem | Probable Cause | Remedy |
|---|--|---|
| The display shows a device fault. | An error has occurred in the device and requires servicing. | Contact your home care provider for service. |
| Symptoms of OSA have recurred. | <ol style="list-style-type: none"> 1. Air filter may be occluded. 2. Sleep apnea condition has changed. | <ol style="list-style-type: none"> 1. Clean or replace the air filter and relocate device away from drapes and other dusty surfaces. 2. Contact your physician. |
| Your skin becomes irritated where the mask touches your face. | <ol style="list-style-type: none"> 1. Headgear is too tight or improperly adjusted. 2. Your mask may not be properly sized or is not the most appropriate shape for you. | <ol style="list-style-type: none"> 1. Loosen the headgear to reduce the contact pressure on your face. 2. Contact your physician or home care provider. |
| Dryness of throat or nose. | Inadequate humidity. | <ol style="list-style-type: none"> 1. Add a DeVilbiss DV5 Heated Humidifier. 2. Increase the heater setting on the humidifier. |
| Condensation of water is collecting in the hose causing a gurgling noise. | <ol style="list-style-type: none"> 1. Your humidification setting is too high. 2. The room temperature fluctuates from higher to lower levels through the night. | Reduce the humidifier heater setting and/or increase the temperature of the room. |
| Air from flow generator seems to be too warm. | <ol style="list-style-type: none"> 1. The air filter(s) is/are dirty. 2. The air inlet port is blocked. 3. The room temperature is too high. 4. The IntelliPAP is located near a heater. | <ol style="list-style-type: none"> 1. Refer to the Cleaning section for instructions on filter maintenance. 2. Unblock the air inlet. 3. Reduce the room temperature. 4. Locate the device away from the heat source so the air being drawn into the device is at room temperature. <p>If the problem persists contact your home care provider.</p> |
| Nasal, sinus or ear pain, possibly with runny nose. | You may have a reaction to air flow pressure. | Discontinue use and contact your physician. |

SPECIFICATIONS

SPECIFICATIONS

| | |
|---|--|
| Size | 4.2" H x 6.5" W x 6.9" D (10.7 cm x 16.5 cm x 17.5 cm) |
| Weight | 2.7 lbs. (1.22 kg) |
| Electrical Requirements..... | 100-240V~, 50/60 Hz |
| Maximum Power Consumption | 65 watts max from AC power source (flow generator only) |
| AutoAdjust CPAP Pressure Range | 3-20 cmH2O |
| Operating Temperature Range..... | 41°F to 104°F (5°C to 40°C) |
| Operating Humidity Range..... | 0% to 95% RH non-condensing |
| Operating Atmospheric Conditions | Sea level to 9,000 feet |
| Storage & Transportation Temperature Range..... | -40°F to 158°F (-40°C to +70°C) |
| Storage & Transportation Humidity Range..... | 0% to 95% RH non-condensing |
| Maximum Limited Pressure | 20 cmH2O under normal use |
| Sound Level (tested per ISO 17510)..... | 26 dBA |

Filter Specifications

- Standard Filter > 3.0 micron particles
- Optional Fine Particle Filter > 0.3 micron particles

Max Flow Rates

- 83 L/m@ 6.5 cmH2O (1/3 max pressure)
- 134 L/m@ 13 cmH2O (2/3 max pressure)
- 163 L/m@ 20 cmH2O (max pressure)
- Pressure Accuracy..... ± 1.0 cmH2O

Pressure Swings

(cmH2O peak to peak at 500mL tidal volume, sine wave profile)

| | | | |
|--------------------|-----|-----|-----|
| Breaths per minute | 10 | 15 | 20 |
| @ 6.5 cmH2O | 0.5 | 0.5 | 0.5 |
| @ 13 cmH2O | 1.0 | 1.0 | 1.0 |
| @ 20 cmH2O | 1.0 | 1.0 | 1.0 |

- Warranty..... Three-year limited
- Equipment classification with respect to protection from electric shock.....Class II
- Degree of protection from electric shock..... Type BF Applied Part
- Degree of protection against ingress of liquids..... IPX I
- Equipment not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- Mode of operation

Clinical Specifications for Digital Outputs

| | |
|------------------------------|------------------------------------|
| Mask Pressure | 0 to 25.5 cmH ₂ O ± 0.5 |
| Patient Flow | -127 to +127 L/min ± 10 L/min |
| Leak Flow | 0 to +127 L/min ± 10 L/min |
| Estimated Tidal Volume | 0 to 1023 mL |

NOTE—Tidal Volume is an estimate. It is provided for trending purposes only.

WARRANTY

The DeVilbiss IntelliPAP device is warranted to be free from defective workmanship and material for a period of three years from date of purchase. Any defective part(s) will be repaired or replaced at DeVilbiss's option if the device has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your DeVilbiss provider or DeVilbiss Customer Service Department for instructions:

U.S.A. 800-338-1988, 814-443-4881

Canada 800-263-3390

Europe 44-138-444-6688.

NOTE—This warranty does not cover providing a loaner device, compensating for costs incurred in rental while said device is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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Sunrise Medical Ltd.
Sunrise Medical Pty. Limited