RESMED

Swift[™] FX for Her

NASAL PILLOWS SYSTEM

Fitting Guide



- Holding the pillows at your nose, pull the headgear over your head.
- Headgear should sit midway between the ears and eves.



- Swift[™] FX for Her pillows are designed to provide a natural fit.
- Minor adjustment can help position the pillows at the most comfortable angle in the nose.



- Adjust the top strap first.
- Pull the strap through the buckle one notch at a time.
- Do not overtighten.



- Connect the Swift FX for Her short tube to the device air tubing.
- Swift FX for Her is now fitted and ready for use.



- Adjust the backstrap to fine-tune the fitting if necessary.
- Do not overtighten.
- The backstrap can be positioned over or under the hair.



 With the device turned on, check that the nasal pillows are properly inflated. To do this, pull the nasal pillows away from the face and cheek, make sure they are inflated and reposition them in the nose.



OPTIONAL SOFT WRAPS

- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on the cheeks, with the fasteners on the outside.


ResMed Corp San Diego, CA, USA +1 858 836 5000 or 1 800 424 0737 (toll free). ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). See www.resmed.com for other ResMed locations worldwide. Swift is a trademark of ResMed Ltd and is registered in the U.S. Patent and Trademark Office. Breathe-O-Prene is a trademark of AccuMED Technologies, Inc. Velcro is a trademark of Velcro Industries B.V. ©2010 ResMed. Specifications may change without notice. 1014015/2 10 08

RESMED

Swift[™] FX for Her

NASAL PILLOWS SYSTEM

of a tube retainer

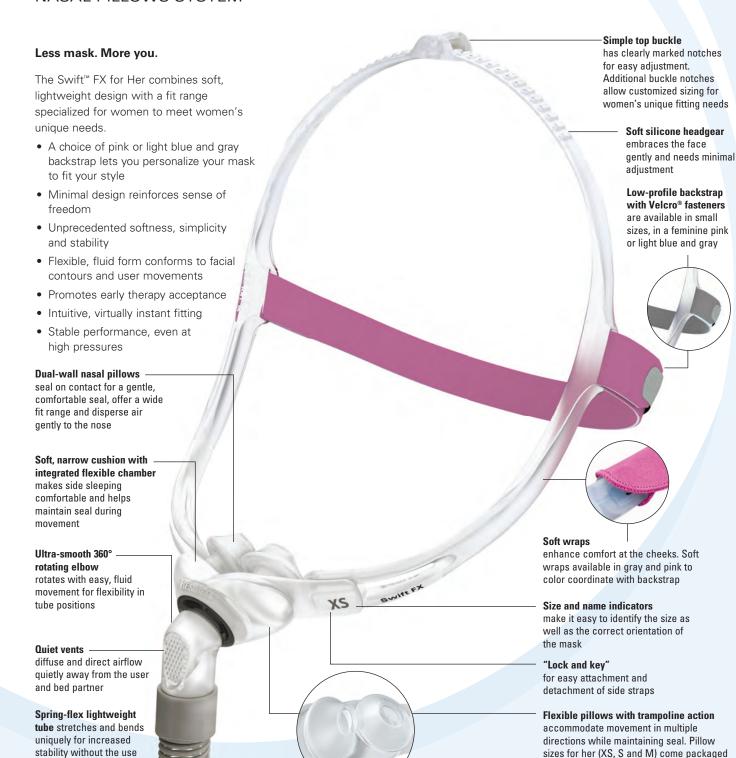

with mask to fit most female patients

STYLE AND DESIGN WITH HER IN MIND



Soft wraps

Added softness at the cheeks enhances comfort and minimizes pressure.

Hair management made easy

Adjustable backstrap can be worn over or under the hair to accommodate a wide range of hairstyles.

The feminine touch

Breathe-O-Prene® headgear material keeps moisture away from skin for maximum comfort. Both pink and light blue and gray backstraps are included in the packaging.







Women and OSA Facts

- Anecdotal evidence suggests that 30 to 50% of new OSA patients are women¹
- The risk of sleep apnea in women after menopause is equal to that of men²
- Menopause may play a role in the severity of sleep apnea³
- Snoring during pregnancy may be an indication of developing sleep apnea⁴
- Insomnia and morning headaches are often the main indicators of sleep apnea in women^{5,6}
- Women who snore are almost twice as likely to have high blood pressure⁷
- Sleep apnea is related to a higher risk of depression in women⁸



Swift FX for Her Nasal Pillows System

(includes 3 pillow sizes: Extra Small, Small and Medium)

Product Code: Americas

61540

- 1 ResMed estimate based on internal data
- 2 Pickett et al. J Appl Physiol 1989
- 3 Resta et al. *Eur J Clin Investig* 2003 4 Charbonneau et al. *Am Rev Respir Dis* 1998
- 5 Shepertycky et al. *Am Rev Respir Dis* 1936
- 6 Ambrogetti et al. *Am New Teal J Med* 1991
- 7 Hu et al. Am J Epidemiol 1999
- 8 Smith et al. Chest 2002
- * Testing per ISO 3744:1994 Acoustics determination of sound power levels of noise using pressure at 10 cm H₂O. Quoted percentage comparisons are calculated by converting sound power values from a logarithmic scale to a linear scale.

Swift[™] **FX** for Her

NASAL PILLOWS SYSTEM

Components Card

Frame assembly Complete system Frame system 61540 (no headgear, no pillows) (no headgear) (includes XS, S and M pillows,* 61528 XS **61510** pink and light blue backstraps, S **61511** and pink and gray soft wraps) M **61512** L 61513 Headgear assembly (includes headgear, pink and light blue backstraps, and pink and gray soft wraps) 61543 Short tube assembly/ frame assembly (includes short tube, elbow and swivel) 61528 (1) Pillows Soft wraps XS 61520 Pink 61544 (2) S 61521 Gray 61530 (2) M 61522 L* 61523 XS Extra Small **S** Small M Medium **L** Large

*Large (L) pillows are available separately