

RESMED

Nothing is like an FX mask.



Featuring a minimal design that reinforces a sense of freedom, the **Swift FX for Her** and **Swift FX Bella** are light, soft and streamlined, so you can look and feel more like yourself.



Swift™ FX for Her
NASAL PILLOWS SYSTEM



Swift™ FX Bella
NASAL PILLOWS SYSTEM

Swift™ FX for Her

More comfort options for increased compliance

The Swift FX for Her nasal pillows system combines a soft, lightweight design with a fit range specialized for women to meet her unique needs. Patients have the option of using the headgear with pink backstrap or Bella loops accessory depending on their style and comfort preferences, giving women more control over their therapy and promoting long-term compliance.

Dual-wall nasal pillows seal on contact for a gentle, comfortable seal, offer a wide fit range and disperse air gently to the nose

Soft, narrow cushion with integrated flexible chamber makes side sleeping comfortable and helps maintain seal during movement

Ultra-smooth 360° rotating elbow rotates with easy, fluid movement for flexibility in tube positions

Quiet vents diffuse and direct airflow quietly away from the user and bed partner

Spring-flex lightweight tube stretches and bends uniquely for increased stability

Simple top buckle has clearly marked notches for easy adjustment.

Soft silicone headgear embraces the face gently and needs minimal adjustment

"Lock and key" for easy attachment and detachment of headgear straps


Size and name indicators make it easy to identify the size as well as the correct orientation of the mask

Low-profile backstrap with Velcro® fasteners

Soft wraps enhance comfort at the cheeks

Flexible pillows with trampoline action accommodate movement in multiple directions while maintaining seal. Pillow sizes for her (XS, S and M) come packaged with mask to fit most female patients

Swift FX for Her components

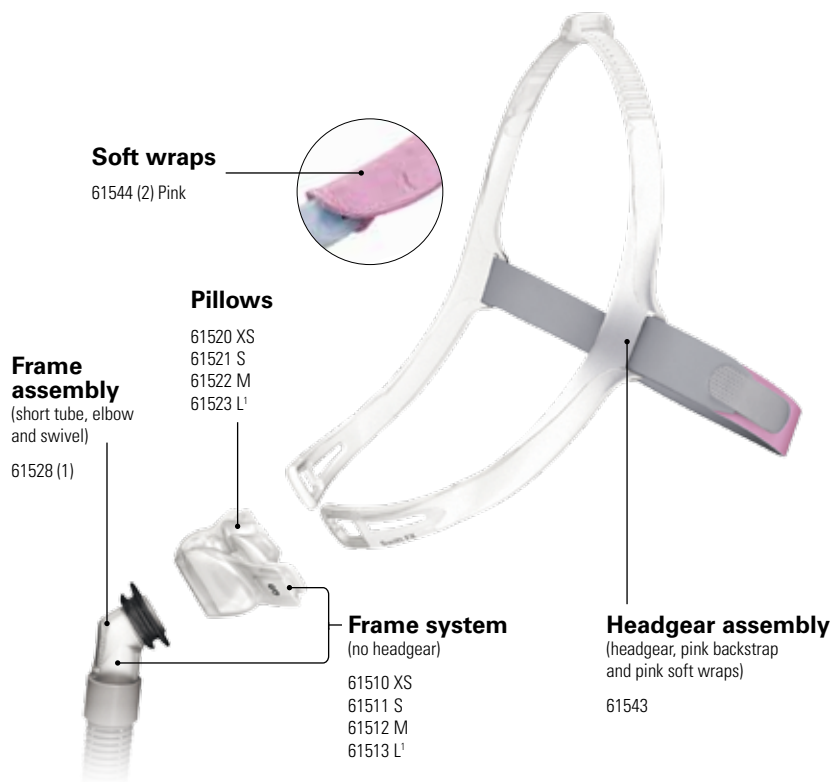

PRODUCT CODES

Complete Systems

Swift FX for Her Nasal Pillows Mask

Style	Part Number
Swift FX for Her	61540
Includes Extra Small (XS), Small (S) and Medium (M) pillows, pink backstrap and pink soft wraps. ¹	

Reimbursement Codes

HCPCS	Code Descriptor
A7034	Mask interface—mask or cannula, 1 per 3 months
A7033	Pillow for use on nasal cannula type interface—replacement, 2 pairs per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

¹ Large (L) pillows are available separately

Swift™ FX for Her
NASAL PILLOWS SYSTEM

Less mask. More you.

Swift™ FX Bella

Style and design with her in mind

The Swift FX Bella nasal pillows system features the exclusive Bella loops accessory in place of the headgear, offering an additional user-friendly way to manage a wide range of hairstyles. It maintains the same comfort features of the original Swift FX for Her to offer the quietest and most comfortable therapy experience.




Dual-wall nasal pillows seal on contact for a gentle, comfortable seal, offer a wide fit range and disperse air gently to the nose

Soft, narrow cushion with integrated flexible chamber makes side sleeping comfortable and helps maintain seal during movement

Ultra-smooth 360° rotating elbow rotates with easy, fluid movement for flexibility in tube positions

Quiet vents diffuse and direct airflow quietly away from the user and bed partner

Spring-flex lightweight tube stretches and bends uniquely for increased stability

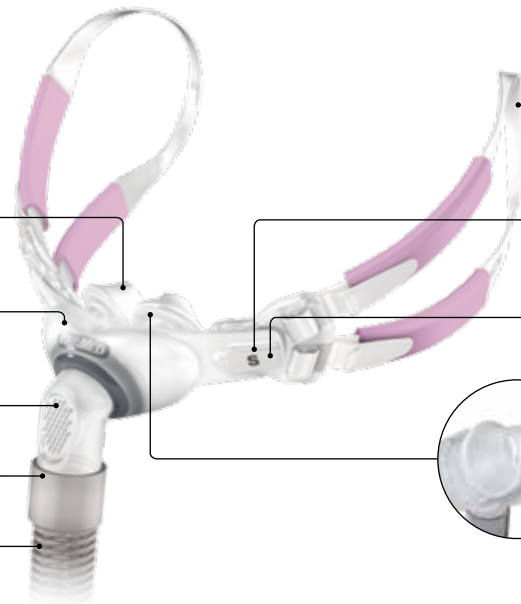


Bella loops fit comfortably around the ears and stay out of her hair for a more streamlined look and feel

Size and name indicators make it easy to identify the size as well as the correct orientation of the mask

“Lock and key” for easy attachment and detachment of headgear straps or bella loops

Flexible pillows with trampoline action accommodate movement in multiple directions while maintaining seal. Pillow sizes for her (XS, S and M) come packaged with mask to fit most female patients

Swift FX Bella components

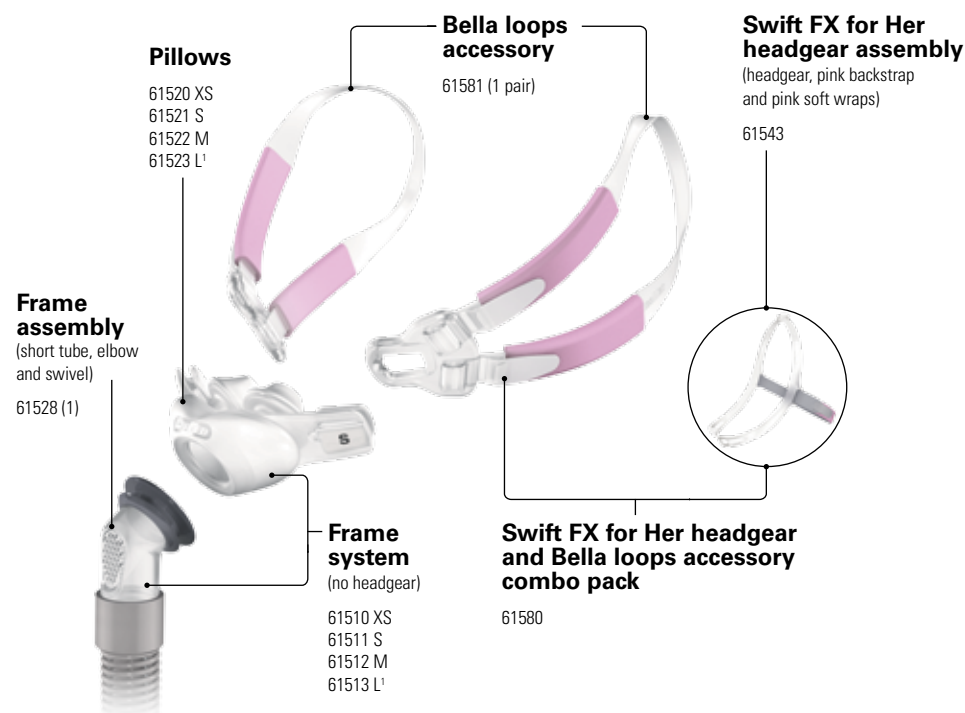

PRODUCT CODES

Complete Systems

Swift FX Bella Nasal Pillows Mask

Style	Part Number
Swift FX Bella	61560

Includes Extra Small (XS), Small (S) and Medium (M) pillows, Bella loops accessory, Swift FX for Her headgear with pink backstrap and pink soft wraps.¹

Reimbursement Codes

HCPCS	Code Descriptor
A7034	Mask interface—mask or cannula, 1 per 3 months
A7033	Pillow for use on nasal cannula type interface—replacement, 2 pairs per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

¹ Large (L) pillows are available separately

Swift™ FX Bella
NASAL PILLOWS SYSTEM

Less mask. More you.

Fitting

Swift FX for Her



- 1
- Holding the pillows at your nose, pull the headgear over your head.
 - Headgear should sit midway between the ears and eyes.



- 2
- Adjust the top strap first.
 - Pull the strap through the buckle one notch at a time.
 - Do not overtighten.



- 3
- Adjust the backstrap to fine-tune the fitting if necessary.
 - Do not overtighten.
 - The backstrap can be positioned over or under the hair.



- 4
- Swift FX pillows are designed to provide a natural fit.
 - Minor adjustment can help position the pillows at the most comfortable angle in the nose.
 - Connect the free end of the short tube assembly to the device air tubing.

Optional soft wraps

- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on the cheeks, with the fasteners on the outside.



Swift FX Bella



- 1
- Hold the pillows at your nose, ensuring the ResMed logo is above the pillows facing up. Pull one strap around and behind your ear.
 - Ensure the strap lies flat and is not twisted.
 - Repeat with the other strap.



- 2
- Adjust the mask fit by pulling the upper straps along the notches until a comfortable fit is achieved. Repeat with the lower straps. Do not overtighten.



- 3
- Connect the free end of the short tube assembly to the device air tubing.

S9™ Series
|More. Comfort.



Combine a Swift FX for Her mask with ResMed's S9 AutoSet™ for Her therapy device for a comfortable, quiet, lifestyle-friendly system

Less mask. More you.