

## Frequently Asked Questions Vacu-Aide® QSU Quiet Suction Unit

**Q1: How long will the battery operate the unit?**

*A1: A fully charged battery will provide approximately 60 minutes of continuous operation at a zero vacuum level (free flow). Operation time will decrease with higher vacuum levels.*

**Q2: What is recommended for the initial charge of the battery?**

*A2: A discharged battery will require 10-17 hours (depending on depth of discharge) of charging to reach a full capacity.*

**Q3: If not in use for an extended period of time are there any recommendations for keeping the battery in good condition?**

*A3: Yes, it's our recommendation that the battery be recharged a minimum of every 3 months.*

**Q4: If traveling internationally is there anything needed to use my Quiet Suction Unit?**

*A4: Yes, the correct power cord must be used, as the unit can operate on any AC voltage.*

**Q5: How do I get replacement accessories and parts for my Quiet Suction Unit?**

*A5: Through an Authorized DeVilbiss Provider, which can be found on our web site under Find a Provider, located at the very top of our web pages.*

**Q6: What is the warranty of my Quiet Suction Unit?**

*A6: 7314 series – 2 years from date of purchase (excludes the battery & collection canister); the internal rechargeable battery is warranted for 90 days.*

**Q7: When should the filter of the disposable canister be replaced?**

- A7:*
- 1. If it becomes wet (it will immediately occlude when wet)*
  - 2. Between patients*
  - 3. When disposing of a used canister (the filter actually becomes a plug for the suction canister when disposed of)*

