Neb-u-Tyke™ Neb-a-Doodle

JOHN BUNN®
A Graham-Field Brand

Nebulizer Compressor Item JB0112-070 User Manual

**Read this manual before operating the nebulizer.
Save this manual for future reference.**

**Note: The most current version of this manual
can be found online at www.grahamfield.com.**

**⚠ Federal Law (USA) restricts this device to sale by or
on the order of a physician. This device should not
be used unless the operator has been instructed by a
qualified healthcare professional.**

| | |
|--|--|
| This product has been tested and successfully approved with the following standards: | UL1431 |
| | EN 60601-1 |
| |  0434 |

CONTENTS

INTRODUCTION 3
 INTENDED USE 3
 SYMBOLS KEY 4
IMPORTANT SAFETY PRECAUTIONS 5
PRODUCT DESCRIPTION 7
 FEATURES AND COMPONENTS 7
SET-UP 8
OPERATION 9
 NEBULIZER OPERATION 9
 DRAWING BOARD OPERATION 10
 SAFETY OVERLOAD 11
MAINTENANCE 11
 CLEANING THE COMPRESSOR'S OUTER CASE 11
 CLEANING THE NEBULIZER CUP 12
 CLEANING THE DRAWING BOARD 12
 REPLACING THE FILTER 12
 STORAGE 13
TROUBLESHOOTING 13
SPECIFICATIONS * 14
LIMITED WARRANTY 15
COMPRESOR DE NEBULIZADOR MANUAL USUARIO (EN ESPAÑOL) 17

Graham-Field and **John Bunn** are registered trademarks, and **Neb-u-Tyke** and **Neb-a-Doodle** are trademarks, of **GF Health Products, Inc.**

Packaging, warranties, products, and specifications are subject to change without notice. GF Health Products, Inc. is not responsible for typographical errors.

INTRODUCTION

This user manual contains important information and safety precautions for the John Bunn Neb-u-Tyke Neb-a-Doodle Nebulizer Compressor.

Before using your Nebulizer Compressor, please read and understand this entire user manual. Take special note of all safety precautions marked DANGER and WARNING. Save this user manual for future reference.

Intended Use

The Neb-u-Tyke Neb-a-Doodle Compressor Nebulizer System is designed to provide a source of compressed air for aerosol therapy. It is used in conjunction with a jet (pneumatic) nebulizer to produce medicated aerosols for inhalation by pediatric and adult patients with respiratory symptoms.

Indications for therapy include asthma, chronic bronchitis, infection of the upper respiratory tract, chronic obstructive pulmonary disease, and other respiratory disorders in accordance with a physician's prescription. Do not use this product for any other purpose than the preceding. This device can be used with adult or pediatric patients under physician's prescription.

The intended use of the Neb-a-Doodle Drawing Board is drawing, writing, and doodling only.

Symbols Key

| | |
|---|--|
| O | Off |
| I | On |
| ~ | Alternating current |
| IPX0 | Enclosed equipment without protection against ingress of water |
|  | Attention (read the instructions) |
| II | Class II (not applicable for UL1431) |
|  | BF: This product is in accordance with the degree of protection against electric shock for type BF equipment. (not applicable for UL1431) |
|  | Disposal of Electrical & Electronic Equipment (WEEE): This product should not be treated as household waste, but submitted to an appropriate collection point for the recycling of electrical and electronic equipment. For more detailed information about the recycling of this product, please contact your local city office, household waste disposal service, or the Graham-Field authorized distributor from whom you purchased this product. |

IMPORTANT SAFETY PRECAUTIONS

When using electrical products, especially when children are present, always follow basic safety precautions, including the following:

- ⚠ DANGER:** Indicates a potential hazard situation or unsafe practice that, if not avoided, will result in death or serious personal injury.
- ⚠ WARNING:** Indicates a potential hazard situation or unsafe practice that, if not avoided, could result in death or serious personal injury.
- ▲ Caution:** Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor personal injury or product/property damage.

Note: Provides application recommendations or other useful information to ensure that you get the most from your product.

READ ALL INSTRUCTIONS BEFORE USING

- ⚠ DANGER:** To reduce the risk of electrocution:
 1. Always unplug this product immediately after use.
 2. Do not use this product while bathing, showering, washing dishes, or close to water sources of any kind.
 3. Do not place or store device where it can fall or be pulled into a tub or sink.
 4. Do not place device in or drop into water or other liquid.
 5. Do not reach for a device that has fallen into water. Unplug it immediately.
- ⚠ DANGER:** To reduce the risk of burns, electrocution, fire or personal injury:
 1. Never leave this device unattended when plugged in.
 2. This product contains small parts that may present a choking hazard to small children, and the tubing may present a strangulation hazard. *Always use close supervision when this device is used by or near children or those who require close supervision.*

3. **Children under the age of 3, or any patient who is unable to use a mouthpiece properly under supervision, should use a mask.**
4. **Use this device only as intended and described in this manual. Do not use attachments or accessories not recommended by GF Health Products, Inc.**
5. **Never operate this product if:**
 - a. **it has a damaged cord or plug,**
 - b. **it is not working properly or it has been dropped or damaged,**
 - c. **it has been dropped into water.**

Return the product to your Graham-Field authorized distributor for examination and repair.


6. **Keep the cord away from heated surfaces.**
7. **Never block the air openings of this product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair and the like.**
8. **Never drop or insert any object into any openings or hose.**
9. **Never use this device while drowsy or sleeping.**
10. **Do not use this device outdoors or operate where aerosol (spray) products are being used.**
11. **Do not use this device in the presence of flammable anesthesia or where oxygen or nitrous oxide is being administered. This device has no AP or APG protection.**
12. **Connect this device to a properly grounded outlet only.**
13. **This device has a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not modify the plug in any way. Do not attempt to defeat this safety feature.**
14. **Unplug the device before filling or cleaning the nebulizer.**
15. **This device is oil-free. Do not lubricate.**
16. **Do not remove the jet nebulizer while the device is operating.**

17. Do not leave medication inside the jet nebulizer when it is not in use.
18. Use the nebulizer intermittently; the maximum operating period is 30 minutes. The nebulizer must stand unused for 30 minutes between uses.
19. The mouthpiece and masks are for use by one person only; in order to avoid infection, do not share them.

PRODUCT DESCRIPTION

Features and Components

Please note nebulizer features and components, labeled in the illustration below.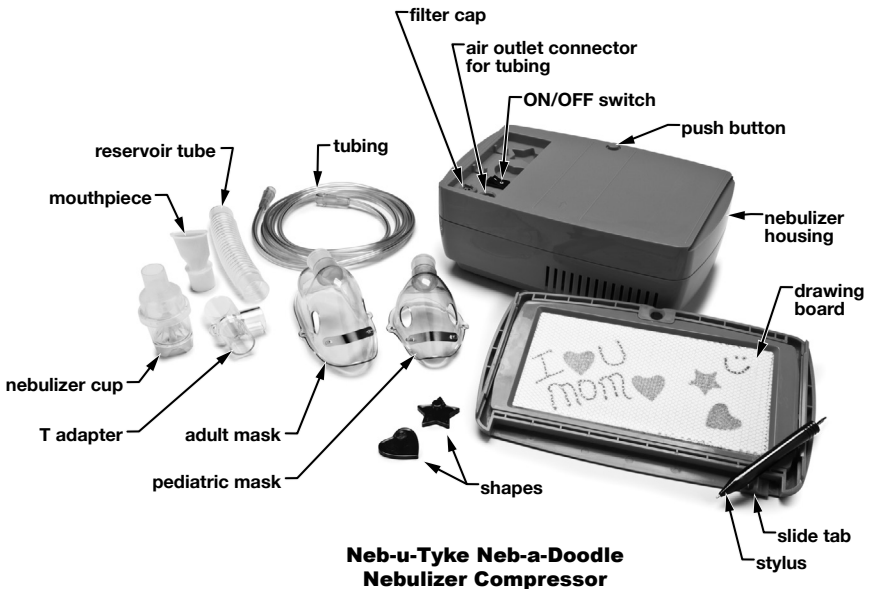

Note: Neb-a-Doodle replacement parts are available at www.grahamfield.com or from your Graham-Field authorized distributor.


SET-UP

*Note: Before initial operation, clean the nebulizer cup assembly following the instructions given in **MAINTENANCE/Cleaning the Nebulizer Cup**.*

⚠ WARNING: Ensure that the nebulizer compressor is unplugged during set-up.

1. Before each use, inspect the compressor and nebulizer cup assembly for damage or wear; replace as needed.
2. Place the compressor on a table or other flat stable surface (not on the floor), in an area that is clean and dust-free. Be sure you can easily reach the controls when seated.
3. Open the nebulizer medication cup: Twist the cap counter-clockwise until it stops, then lift cap off.
4. Pour the prescribed medication into the nebulizer medication cup. Do not fill with more than 5 ml per treatment.


5. Install cap on cup. Turn clockwise to tighten.

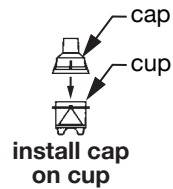6. If using the mouthpiece for treatment, proceed to step 7. If using a mask, install the bottom part of the mask directly onto the top of the nebulizer medication cup cap as shown at right. Proceed to step 9.


7. Insert mouthpiece into large end of T adapter.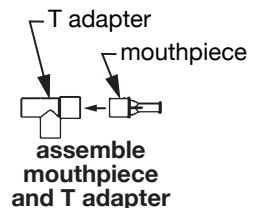

Note: The reservoir tube, shown in the illustration earlier in this manual, acts as a holding tank for the mist, reducing the amount lost during treatment. Its use is optional. If you wish to use the reservoir tube, install it now on the small end of the T adapter.

8. Install T adapter onto top of nebulizer medication cup cap.
9. Connect the tubing to the adapter on the bottom of the nebulizer medication cup.
10. Connect the tubing's other end to the compressor air outlet connector.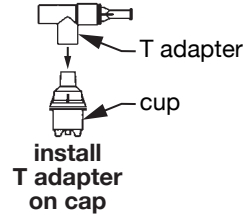

⚠ WARNING: Before use, check that all parts are securely connected. Use of an improperly assembled nebulizer could diminish or prevent adequate delivery of medication and the effectiveness of the treatment.

OPERATION

Note: The drawing board must be removed from the nebulizer housing before use. To remove the drawing board, push the button and slide the drawing board in the opposite direction from the cover arrow.

Nebulizer Operation

▲ Caution: The Neb-u-Tyke Neb-a-Doodle Nebulizer Compressor is designed for intermittent use only. Do not operate it continuously for more than 30 minutes for a single use without turning it off and allowing a cooling period of least 30 minutes.

1. Ensure that the ON/OFF switch is in the OFF position.
2. Plug the power cord into a properly grounded electrical wall outlet.
3. Move the ON/OFF switch to the ON position to start the compressor.
4. Place the nebulizer mouthpiece between teeth and close mouth (or install the mask on face).

5. As aerosol starts flowing, inhale deeply and slowly through mouthpiece (or mask).


⚠️ WARNING: Never use a nebulizer with a clogged nozzle. If the nozzle is clogged, the aerosol mist will be reduced, altered or stopped, which will diminish or eliminate the effectiveness of the treatment. If clogging occurs, replace the nebulizer.

Note If you need to interrupt treatment, move the ON/OFF switch to the OFF position.

6. When the treatment is complete, move the ON/OFF switch to the OFF position to turn off the compressor.
7. Unplug the compressor from power outlet.
8. Disconnect nebulizer from tubing.
9. Clean nebulizer as instructed in **MAINTENANCE/Cleaning the Nebulizer Cup**.

Drawing Board Operation

To remove the drawing board, push the button and slide the drawing board in the opposite direction from the cover arrow.

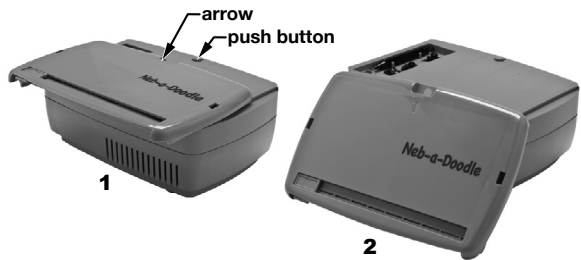To install the drawing board, insert the drawing board on the side with the housing arrow that matches the cover arrow, then push it all the way to secure it to the lock button. This ensures the drawing board stays in place. To prevent damage to the unit, ensure you do not insert it from the other direction.

To draw on the drawing board, move the included stylus pen, pointed end down, along the drawing board. To use the included shapes, place them on the drawing board and press lightly.

To erase the drawing board, move the slide tab over the drawing board.

⚠️ WARNING: This product contains small parts that may present a choking hazard to small children. *Always use close supervision when this device is used by or near children or those who require close supervision.*

Safety Overload

Note: Do not exceed 30 minutes of continuous operation.

1. The motor of this device has a built-in thermal overload protector. Should the motor overheat, the protector will automatically shut off the motor. Should this occur, move the ON/OFF switch to the OFF position and allow the motor to cool down for approximately 30 minutes before turning it on again.
2. If the overload protector shuts off the motor frequently, you may have an unstable voltage situation.
3. If the device shuts down and cannot restart, it may need to be replaced. Call your Graham-Field authorized distributor immediately.

MAINTENANCE

Cleaning the Compressor's Outer Case

⚠️ WARNING: Electrical shock hazard. Do not remove the outer case of the device. All disassembly and maintenance of the device must be performed by a qualified service technician. Refer service to qualified service personnel.

⚠️ WARNING: This device does not require oil. Do not attempt to lubricate any internal parts.

⚠️ WARNING: Unplug device before cleaning. Do not submerge in water for cleaning.

Wipe the compressor's outer case with a clean, damp cloth weekly to keep it dust-free. Do not use any powdered type cleaners or soap.

Cleaning the Nebulizer Cup

All parts of the disposable nebulizer, except tubing, should be thoroughly cleaned after each use. If your physician specifies a specific cleaning procedure, follow that procedure.

⚠ WARNING: To prevent possible risk of infection from contaminated medication, clean the nebulizer after each treatment.

1. After each use, disassemble the nebulizer cup, T adapter and mouthpiece completely, wash all parts in warm soapy water, and rinse thoroughly. Allow to air dry. Store in a clean, dry location.
2. Once daily, after washing as above, soak nebulizer components in a fresh solution consisting of one part warm water and one part white vinegar for 30 minutes followed by a warm water rinse. Allow to air dry. Store in a clean, dry location.

Cleaning the Drawing Board

Wipe the drawing board outside cover with a clean, damp cloth weekly. Do not use any liquid, cleansers or soap on any surface of the drawing board.

Replacing the Filter

Note: Change the filter once every 30 days or when the filter changes color. Additional filters can be purchased at www.grahamfield.com or from your Graham-Field authorized distributor.

1. Ensure that ON/OFF switch is in the OFF position. Open the filter cap located next to the ON/OFF switch.
2. Remove the dirty filter with a small, pointed object. Discard the filter.
3. Replace with a clean filter.
4. Replace the filter cap.

▲ Caution: Do not use cotton or other materials as a filter.

▲ **Caution: Do not wash or clean the filter.**

▲ **Caution: Do not reuse the filter. Do not operate compressor without a filter. Compressor damage can result.**

▲ **Caution: The disposable nebulizer kit is for single patient use only.**

Storage

Keep the device and accessories dry. Avoid exposure to direct sunlight.

TROUBLESHOOTING

If your Neb-u-Tyke Neb-a-Doodle Nebulizer Compressor fails to function, consult the following Troubleshooting guide. If the problem persists, consult your Graham-Field authorized distributor.

| Problem | Cause and Solution |
|------------------------|--|
| Device doesn't operate | Check if plug is properly fit into a grounded electrical outlet. |
| | When device has been run continuously for more than 30 minutes just before using, auto-shut-down may have been activated by the built-in thermal protector; ensure that device is OFF for at least 30 minutes before next usage. |
| Nebulization is weak | Check for proper electrical voltage. |
| | Check tubing for blockage or air leakage at connection to Neb-u-Tyke compressor or nebulizer cup; replace as needed. |
| | Check that the nebulizer cup is properly assembled and not damaged. If there is any damage, replace as needed. |
| | Check if filter is dirty; replace filter as needed. |

SPECIFICATIONS *

| Specification | Description | |
|---------------------------|--|---|
| Max pressure (compressor) | 32.5 PSI | |
| Max flow rate | Approximately 12 LPM | |
| Nebulization rate | .33 ml/min | |
| Particle dimension | .5 - 5 microns | |
| Capacity | 5 ml | |
| Filter | Sponge | |
| Dimensions (LxWxH) | 10" x 6.5" x 4.2" (255 mm x 166 mm x 108 mm) | |
| Weight | 4 lb (1860 g) | |
| Electrical Requirement | 115VAC/60Hz | |
| Current | 1.2A (rate) | |
| Fuse | 2.5A 250V | |
| Power Cord | Black, 7.5 ft (90") / 2,286 mm | |
| Plug | 2-prong type | |
| Sound level | 56.5 dBA | |
| Environment | Temperature | Operation: 41°F to 104°F (5°C to 40°C) |
| | | Storage: -40°F to 158°F (-40°C to 70°C) |
| | Humidity | Operation: 30% to 95% RH |
| | | Storage: 10% to 95% RH |

* Specifications are subject to change without notice.

LIMITED WARRANTY

GF Health Products Inc. warrants this product to be free from manufacturing defects in material and workmanship for its period of warranty that is in accordance with industry standards. This warranty is extended only to the original purchaser/consumer or dealer/non-consumer of this new product and to no other purchaser or transferee.

GF Health Products, Inc. warrants the John Bunn JB0112-070 Nebulizer Compressor and its components to be free from defects in workmanship and materials as listed below:

Compressor: Five (5) years

Accessories: Six (6) months

The Warranty period for the consumer commences on the first date a product is delivered to consumer by seller/dealer. If the product is rented or leased, the warranty period commences on the invoice date from *GF Health Products, Inc.* A copy of the invoice showing date of purchase must be provided when submitting warranty claims. When proof of purchase date is not provided, warranty coverage shall commence upon *GF Health Products, Inc.*'s invoice date to the dealer/purchaser.

If within the warranty period, the product or component part is proven to *GF Health Products, Inc.*'s satisfaction to be defective, *GF Health Products, Inc.* shall provide, at its option, one of the following: (1) repair or replacement of any defective or nonconforming part or product or (2) a credit and/or refund of the original selling price. GF HEALTH PRODUCTS, INC.'S SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO SUCH REPAIR, REPLACEMENT, CREDIT AND/OR REFUND. This warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to the manufacturer.

For warranty service, please contact the authorized distributor from whom you acquired your *GF Health Products, Inc.*

product. Upon receiving notice of an alleged defect in a product, *GF Health Products, Inc.* will issue a return authorization. The defective product or part(s) must then be returned, at the purchaser's cost, for warranty inspection using the serial number as identification (or, if the product is not serialized, lot number and date code) within thirty (30) days of return authorization issue date. In the event you do not receive satisfactory warranty service, please contact *GF Health Products, Inc.* at the address below. DO NOT return products to our factory without prior authorization.

LIMITATIONS AND EXCLUSIONS: The foregoing warranty excludes consumables, such as filters and tubing, as these require routine maintenance. The foregoing warranty shall not apply to serial numbered products if the serial number has been removed or defaced. Products subjected to negligence, abuse, misuse, improper operation, improper maintenance, improper cleaning, improper storage, or damages beyond *GF Health Products, Inc.*'s control are not covered by this warranty, and that evaluation shall be solely determined by *GF Health Products, Inc.* This warranty shall not apply to problems arising from normal wear and tear or failure to follow instructions. The warranty shall also not apply to products modified without *GF Health Products, Inc.*'s express written consent; nor shall it apply if parts not manufactured by *GF Health Products, Inc.*, or if parts not complying with original equipment specifications are added to *GF Health Products, Inc.* products, or if the product or part is serviced by an entity not authorized by *GF Health Products, Inc.*

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES AND IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN,

AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO THE REPAIR, REPLACEMENT, CREDIT AND/OR REFUND OF THE DEFECTIVE PRODUCT OR PART PURSUANT TO THE TERMS CONTAINED HEREIN. GF HEALTH PRODUCTS, INC. SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state (province to province). Some states (provinces) do not allow the exclusion or limitation of incidental or consequential damage, or limitation on how long an implied warranty lasts, so the above exclusion and limitations may not apply to you.

This warranty contains the entire agreement between the parties and supersedes any prior, contrary or additional representations or understandings, whether oral or written, concerning our warranty policy.

GF Health Products, Inc. (“Graham-Field”)
2935 Northeast Parkway
Atlanta, GA 30360
Tel 800-347-5678, Fax 800-726-0601

USA Corporate Headquarters:

**GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360
telephone: 800-347-5678, 770-447-1609
fax: 800-726-0601, 678-291-3232**

www.grahamfield.com

Made in China

**© August 2009 GF Health Products, Inc.
JB0112-070-INS-LAB-RevA09**

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